## ClinChek® - Control Serum Control lyophilised / Kontrollserum lyophilisiert

### FOR ANTIPILEPTICS / FÜR ANTIPILEPTIKA

#### Intended use:

ClinChek® serum controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on calf serum and are available with mean values in the therapeutic as well as in the elevated range. The controls have to be prepared like patient samples in one series of analyses.

#### Reconstitution:

Add exactly 3.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

If it is necessary to have a defined buffer-constitution of the control for further sample preparation (e.g. for pH-adjustment), you can also dissolve the lyophilised serum control directly in the appropriate buffer.

#### Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised serum control is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is:

- at least 24 hours when stored at 15 - 30 °C
- at least 10 days when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C (avoid repeated freezing and thawing)

#### Notes:

Routine analyses of the controls are performed according to the test procedure for patient samples. The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.5 %. The average residual moisture of this lot is 1.35 %.

#### Mean values:

The mean values and confidence intervals were established in additional independent reference laboratories with supervision of RECIPE, according to the guideline of the German Medical Council for quality assurance (RiliBäk).

#### Pack size:

ClinChek® Serum Control  
Level I  
10 x 3 ml, **order no.: 15080**  
Level II  
10 x 3 ml, **order no.: 15081**  
Level I, II  
2 x 5 x 3 ml, **order no.: 15082**

#### Precautions:

The calf serum which was used for manufacturing the serum controls should be considered as potentially infectious and therefore the controls should be treated with appropriate care.

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In Germany, Phenobarbital, Phenytoin, Carbamazepine and Valproic Acid are subject to the guideline of the German Medical Council for quality assurance. For assessment of the internal quality control, please also refer to table B1a of the guideline.

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RECIPE CHEMICALS + INSTRUMENTS GmbH