ClinChek® - Control Whole Blood Control lyophilised / Vollblutkontrolle lyophilisiert

FOR CYCLOSPORINE A / FÜR CYCLOSPORIN A

Intended use:

ClinChek® whole blood controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human blood and are available with mean values in the elevated range as well as in the very high range, comparable to concentrations found during C2-level-monitoring. The controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 2.0 ml of HPLC-water to the vial. To dissolve the material completely, we recommend a so-called roller mixer. A treatment of 30 to 60 minutes on such an instrument is sufficient.

If it is necessary to have a defined buffer-constitution of the control for further sample preparation (e.g. for pH-adjustment), you can also dissolve the lyophilised whole blood control directly in the appropriate buffer.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised whole blood control is stable for 36 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analyte is:

- at least 8 hours when stored at 15 - 30 °C
- at least 7 days when stored at 2 - 8 °C
- at least 30 days when stored below -18 °C (avoid repeated freezing and thawing)

Notes:

Routine analyses of the controls are performed according to the test procedure for patient samples. The concentrations of the analyte are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.25 %. The average residual moisture of this lot is 0.38 %.

Mean values:

The mean values and confidence intervals were established in additional independent reference laboratories with supervision of RECIPE, according to the guideline of the German Medical Council (RililBäk) with statistical methods. The control range corresponds to the mean value +/- triple standard deviation.

Pack size:

ClinChek® Whole Blood Control for Cyclosporine A

Level IV and V

2 x 2 x 2 ml, order no.: 8903

Precautions:

The human whole blood which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV- and HCV-genome (PCR). Nevertheless, the controls should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH